

Medibin ABV
Anweisung für das sachgerechte Verschließen
Instruction for the appropriate sealing



Medibin ABV 15l / 30l / 60l

Einwegbehälter für anatomisch-pathologische Abfälle
Disposable container for anatomical-pathological waste

Anweisung für das sachgerechte Verschließen

Deckel und Behälter sind vor Verwendung mindestens 24 Stunden in durchschnittlicher Raumtemperatur zu lagern

Der Deckel ist bei Raumtemperatur auf den Behälter aufzusetzen.

An einer Stelle beginnend mit dem Handballen fest aufdrücken, so dass die Deckelkante über den Behälterrand schnappt.

Hiermit dann um den gesamten Behälterumfang fortfahren.

Anschließend visuell oder per Abtasten prüfen, ob die Deckelkante an allen Stellen über den Behälterrand geschnappt ist.

Instruction for the appropriate sealing

Cover and container must be stored in a well-heated area with regular room temperature for at least 24 hours before use.

The cover has to be put on the bin.

Starting at one point of the edge of the bin, the cover has to be fixed by pressing the heel of hand on it strongly until it snaps onto it.

This has to be carried on for the whole extent of the bin.

Afterwards it has to be checked visually or by feeling whether the cover has snapped over the edge of the bin at all points.